2-STAGE MANUAL BREAST PUMP

MODEL GM-20

USER'S MANUAL

dr-frei.com

EN 2-STAGE MANUAL BREAST PUMP

Model GM-20

User's manual (4-8)

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**2-STAGE MANUAL BREAST PUMP
MODEL GM-20
USER'S MANUAL**

Thank you for choosing **Dr. Frei®** Manual Breast Pump GM-20. We are sure that having appraised worthily the high quality and reliability of this device you will become a regular user of the products of **Dr. Frei®** Trade Mark.

Before starting to use the Breast Pump please study the instruction book carefully. Following the rules stated in the user's manual will provide for the faultless operation of the Breast Pump for a long period of time.


TABLE OF CONTENTS

Components List.....	5
HOW TO ASSEMBLE MANUAL BREAST PUMP.....	5
HOW TO USE	6
2-Stage Breast Milk Expression	6
Notice	7
HINTS	7
Breast Milk Storage	8
Warm Tips	8

Components List

1. Soft breast shield
2. Funnel unit
3. Suction knob
4. Diaphragm
5. Stem
6. Dustproof cover
7. Handle knob
8. Handle
9. Adaptor
10. Milk bottle
11. Valve membrane
12. Bottle stand
13. Cap of milk bottle
14. Screw cover
15. Laying
16. Nipple

* The specifications and the outer design are subject to change for improvement reasons without previous notice.

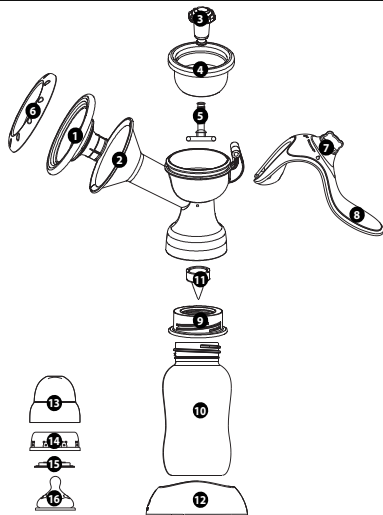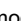
HOW TO ASSEMBLE MANUAL BREAST PUMP

1. Insert the silicone breast shield (1) by aligning the contours of shield to that of the funnel unit (2).
2. Please insert the stem (5) into the silicone diaphragm (4).
3. Fix the suction knob (3) on stem (5).
4. Insert the silicone diaphragm (4) into the funnel unit (2) as shown. Ensure these two parts are filled together.
5. Slide the stem (5) into the fork of handle (8), and then assemble the handle on the pivot

- of funnel unit (2).
6. Push the valve membrane (11) into the pump unit (2).
 7. Screw the bottle (10) with adaptor A (9) and assembled to funnel unit (2).
 8. Please the assembled bottle (10) onto the bottle stand (12).

* Ensure parts are assembled correctly.

HOW TO USE

1. Clean all the parts of breast pump, using the steam sterilizer, wash your hand and assemble all the parts properly, tightened to avoid leakage.
2. Before start sucking milk, gently massage the breast or use a warm towel to compress the breast, which is helpful for extracting breast milk.
3. Sit on a chair relaxed, and tilt your body forward slightly.
4. Press the breast shield gently and tightly against your breast to create a proper seal.
5. At first turn the handle knob to «stimulation» mode «» and press the handle often for stimulation let-down of milk (image 1). Then turn the handle knob often to «sucking» mode «» and press the handle seldom for providing efficient and fast expression (image 2).
6. As per your feeling, turn the adjustable knob in clockwise direction for strong suction power, otherwise for weak power (image 3).
7. Use proper strength for handle operation.
8. After use, please disassemble the container and seal with its cover or cap. It's better to place bottle on bottle base in case of overturning.
9. Please disassemble and clean some parts after use.

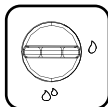
image 1

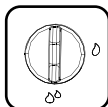
image 2


image 3

PLEASE NOTE: If you can not draw any milk from the breast after five minutes of operation, discontinue use and try again later.

2-Stage Breast Milk Expression


It was found out according to the last deep investigation of breastfeeding physiology,

that the infant, first, suckles the breast often but not strong, it stimulates let-down of milk (stage of «stimulation»). Then, when milk appears, the infant begins to suckle slower and deeper (stage of «suckle»). This discovery led to developing of two-stage expression and appearing of new devices for expression of breast milk. Mechanical breast-pump GM-20 belongs to such devices.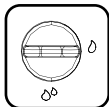

Stage of «stimulation»

Turn the handle knob to mode «S» and press the handle often for stimulation let-down of milk.

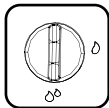Stage of «suckling»

Turn the handle knob to mode «L» and press the handle seldom for providing efficient and fast expression.

Notice

1. All parts should be cleaned and sterilized using the steam sterilizer. Keep it dry and sealed.
2. Do NOT use the device when it is not attached to the bottle because of possible damage.
3. Do not put in direct sunlight.
4. If you have any discomfort or painful feeling after use, consult your Physician or Lactation Consultant.
5. This product is for single use only, if used by more than one person may cause a health risk.

HINTS

When Do You Need To Use Breast Pump?

1. When insufficient breast milk, use it to imitate baby's suction so that promote milk excreting.

2. When too much breast milk, use it to suck away redundant milk for fear breast milk filling up.
3. When facing the pain of bulgy breast, use it to comfort yourself.
4. When wanting to keep breast milk feeding after you go to work, use it and store the breast milk for your baby.

How To Use The Breast Pump Successfully?

1. Choose the time when you are idle and impossible to be disturbed.
2. Having baby's photo with you, to promote milk excreting.
3. Before sucking milk, massage your breast gently or apply your breast with hot towel for a few minutes, it is helpful to excrete milk.
4. Warming and relaxing are helpful for excreting breast milk, you can try to use it after taking a shower.
5. When baby is sucking at one breast, the other one is stimulated and excrete breast milk; it is suggested to use it at this moment.

Breast Milk Storage

If the breast milk would not be used for feeding baby immediately, it must be stored in the feeding bottle and preserved by freezing in the refrigerator. If breast milk to be stored more than 48 hours, It must be rapidly frozen, and use of dedicated storage containers, DO NOT just use a feeding bottle. If you need long time storage of breast milk, we recommend consulting your doctors or breast-feeding professionals, they will provide specialized storage ways.

Warm Tips

Modern people know that Breast Feeding has many unique advantages and adds to the babies healthy development. The devise can be used to extend feeding time and or used for later feeding by the working Mother. We sincerely hope you utilize the Manual Breast Pump to its full advantage, and make nursing your child easier and more pleasant.

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Dr. Frei is a registered Trademark of FreiCare Swiss AG, Seepark 6,
■ Zhejiang Huijun Infant And Child Articles Co., Ltd. No.1 Sheng'an Road, Shangsheng