SEMI-AUTOMATIC BLOOD PRESSURE MONITOR

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DEAR DR. FREI SEMI-AUTOMATIC BLOOD PRESSURE MONITOR OWNER

Thank you for choosing Semi-automatic Blood Pressure Monitor **Dr. Frei®TM** model M-150S. We are sure that having appraised worthily the high quality and reliability of this device you will become a regular user of the products of Swiss Trademark **Dr. Frei®**.

Before starting to use this device please study the user's manual carefully. The user's manual offers all information you need to measure your blood pressure and pulse correctly. For all questions concerning the device please contact your local distributor or **Dr. Frei®** service center in your country.

ATTENTION

This blood pressure monitor is designed to carry out self-control over blood pressure but NOT to make self-diagnosis of hypertension/hypotension. Please DO NOT diagnose by yourselves basing on the measurement results obtained with the blood pressure monitor. Please DO NOT execute self-treatment of high/low blood pressure and DO NOT change the methods prescribed without consulting your doctor.

Type BF applied part.

Read the instructions carefully before using this device.


TABLE OF CONTENTS

IMPORTANT INFORMATION ON BLOOD PRESSURE AND ITS MEASUREMENT.....	6
What Is Blood Pressure?.....	6
Which Values Are Normal?.....	7
ADVANTAGES OF SEMI-AUTOMATIC BLOOD PRESSURE MONITOR MODEL M-150S	8
Blood Pressure Rate Indicator.....	8
Irregular Heartbeat Detector.....	8
GETTING READY FOR MEASUREMENT.....	9
Safety Precautions	9
Components of Your Blood Pressure Monitor Kit.....	9
Description of the Blood Pressure Monitor	10
Inserting the Batteries.....	10
Cuff Connection	11
MEASUREMENT PROCEDURE.....	11
Fitting the Cuff	11
Measuring Procedure	12
Reading Measurement Results	13
Discontinuing a Measurement.....	13
MEMORY FUNCTION	14
ERROR MESSAGES/TROUBLESHOOTING.....	14
CARE AND MAINTENANCE.....	15
SYMBOL DESCRIPTION	16
TECHNICAL SPECIFICATIONS	17
WARRANTY	17

IMPORTANT INFORMATION ON BLOOD PRESSURE AND ITS MEASUREMENT


What Is Blood Pressure?

Blood pressure is the pressure that your blood extends to the vascular walls. Blood pressure is necessary to provide for constant blood flow inside the body. Thanks to it the cells get oxygen that provides for their normal functioning. The heart performs the function of a «pump», sending blood to the blood vessels. Each heart beat creates a certain level of the blood pressure.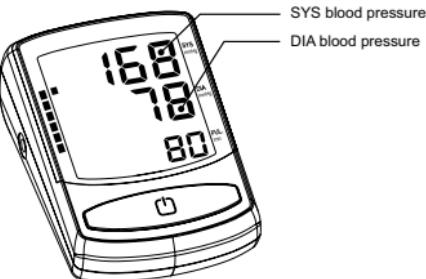

There are 2 kinds of blood pressure: a systolic (upper) one, which corresponds to the heartbeat pushing blood into the arteriae; and a diastolic (lower) one, which means the blood pressure between two heartbeats. Blood pressure is subject to fluctuations during the day even in healthy people. The fluctuations are influenced by a number of factors – time of day, person's condition, physical or mental activity, environment, etc.

Day-Night Fluctuations of the Blood Pressure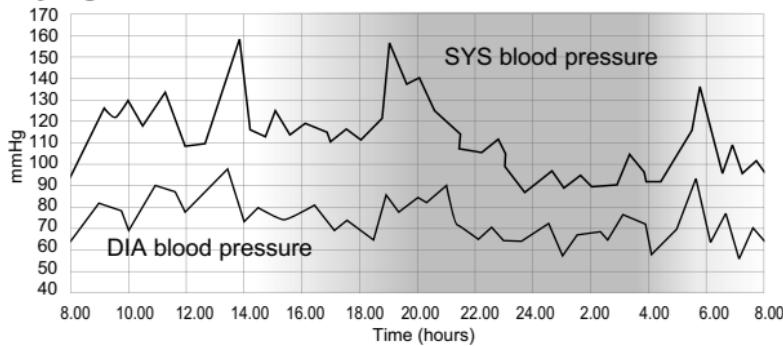

An increase of blood pressure increases the burden onto the heart, affects blood vessels making their walls thick and less elastic.

One of the features of the hypertension is its ability to remain unnoticed for the patient at its early stages. That's why the self-control of the blood pressure is so important. With the illness progressing, headaches and regular dizziness appear, the sight declines, the functioning of vitalises (encephalon, heart, kidneys, blood vessels) breaks down. Without special treatment the complications of hypertension might be kidney damages, breast-pang, paralytic stroke, aphasia, dementia, heart attack and stroke.

Which Values are Normal?

The world standard as for the norms of the blood pressure is the Classification* of the World Health Organization (WHO):

Blood Pressure Category	Systolic Blood Pressure (mmHg)	Diastolic Blood Pressure (mmHg)
Blood pressure too low	< 100	< 60
Blood pressure optimum	100 - 119	60 - 79
Blood pressure normal	120 - 129	80 - 84
Blood pressure slightly high	130 - 139	85 - 89
Blood pressure too high	140 - 159	90 - 99
Blood pressure far too high	160 - 179	100 - 109
Blood pressure dangerously high	≥ 180	≥ 110

* printed with curtailments

- The diagnosis of hypertension requires from the patient to combine medical treatment prescribed by the doctor and mode of life correction.
- People with normal pressure and high normal pressure are recommended to carry out self-control of their tension in order to timely take measures to decrease the blood pressure level down to the optimal one without using any medications.
- For people more than 50 years old high level of systolic blood pressure (higher than 140 mmHg) is more crucial than diastolic pressure.
- Even with blood pressure being normal, people run the bigger risk of hypertension development with advancing age.

ATTENTION

If you have normal results of blood pressure measured under calm conditions but your results are excessively high when measured under the conditions of physical or mental exhaustion, this might be a sign of so called brittle (that is unstable) hypertension. If you suspect that, please consult your doctor.

When measured correctly, if diastolic blood pressure is more than 120 mmHg, it is necessary to call the doctor immediately.

ADVANTAGES OF SEMI-AUTOMATIC BLOOD PRESSURE MONITOR MODEL M-150S

Blood Pressure Rate Indicator

Blood pressure rate indicator is located along the left side in the display. The classification corresponds to WHO (World Health Organization) ranges described in the table of the section «Which values are normal?». After the measurement there appears the color bars in the left part of the display: green zone - optimal blood pressure, yellow - elevated, orange - too high, red - dangerously high. This function helps you to self-orient in the measurement results.

Irregular Heartbeat Detector

This technology indicates allorhythmic heartbeating. If the symbol of IHB  appears on the display that means that certain abnormality in heart beat frequency was detected during the measurement.

In this case, the result may deviate from your normal blood pressure - repeat the measurement. In most cases, this is no cause for concern. However, if the symbol  appears on a regular basis (e.g. several times a week with measurements taken daily) we advise you to tell your doctor. Please show your doctor the following explanation:

Information for the doctor on frequent appearance of the Irregular Heartbeat Detector.

This instrument is an oscillometric blood pressure monitor that also analyses pulse frequency during measurement. The symbol  is displayed after the measurement, if pulse irregularities occur during measurement.

If the symbol  appears more frequently (e.g. several times per week on measurements performed daily) we recommend the patient to seek medical advice. The instrument does not replace a cardiac examination, but serves to detect pulse irregularities at an early stage.

GETTING READY FOR MEASUREMENT

Safety Precautions

1. Use this device ONLY for the intended use described in this manual.
2. Do NOT use accessories which are not specified by the manufacturer.
3. Do NOT use the device if it is not working properly or damaged.
4. Do NOT use the equipment where aerosol sprays are being used, or where oxygen is being administered.
5. Do NOT use under any circumstances on newborns.
6. This device does NOT serve as a cure for any symptoms or diseases. The data measured are for reference only. Always consult your doctor to have the results interpreted.
7. Keep the equipment and its flexible cord away from hot surfaces.
8. Do NOT apply the cuff to areas other than the place directed.
9. Proper maintenance and periodically calibration are essential to the longevity of your device. If you are concerned about your accuracy of measurement, please contact local customer service for help.
10. This system is intended for individuals age 18 or above.
11. Do not use it for diagnosis of hypertension or testing on newborns, babies, young children or persons who cannot express their consent.
12. If you have been diagnosed with arrhythmia or irregular heartbeat, atrial or ventricular premature beats or atrial fibrillation, measurements made with this instrument should only be evaluated after consultation with the doctor or your healthcare professionals.


KEEP THIS MANUAL DURING THE WHOLE LIFETIME OF THE DEVICE

Components of Your Blood Pressure Monitor Kit


The blood pressure monitor kit includes semi-automatic blood pressure monitor model M-150S, the cuff with the hose and pumping bulb, 4 batteries (AA), user's manual, warranty card, kit box.

Description of the Blood Pressure Monitor

Measuring unit Model M-150S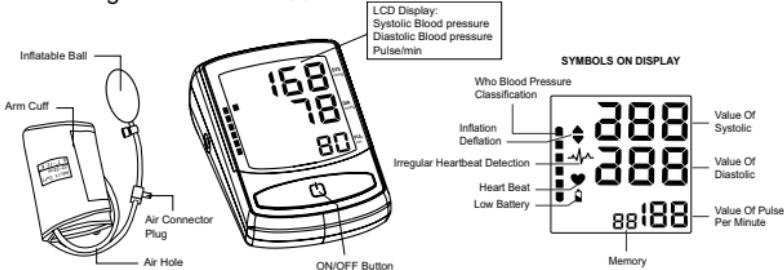


Accessories:

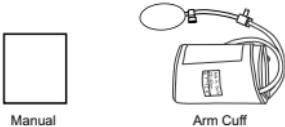Inserting the Batteries

1. Remove the battery cover.
2. Insert new batteries into the battery compartment as shown, taking care that the polarities (+) and (-) are correct.
3. Close the battery cover. Use only LR6, AA batteries.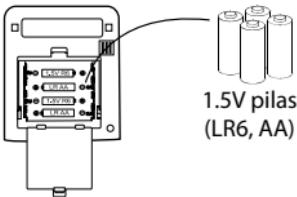


CAUTION

- Insert the batteries as shown in the battery compartment.
- When blinks in the display, replace all batteries with new ones. Do not mix old and new batteries. It may shorten the battery life, or cause the device to malfunction.
- does not appear when the batteries run out.
- Battery life varies with the ambient temperature and may be shorter at low temperatures.
- Remove the batteries if the device is not to be used for a long time.
- The batteries may leak and cause a malfunction.
- Use the specified batteries only. The batteries provided with the device are for testing monitor performance and may have a shorter life.

Cuff Connection

Insert the cuff tube into the opening provided on the side of the device as shown in the picture.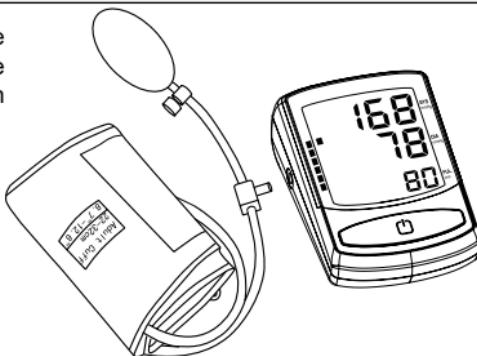

MEASUREMENT PROCEDURE

ATTENTION


- Find time to relax by sitting in a quiet atmosphere for some time before measurement.
- Efforts by the patient to support the arm can increase the blood pressure. Make sure you are in a comfortable, relaxed position and do not flex in the arm during measurement.
- Always measure on the same arm (normally left).
- Remove any garment that fits closely to your upper arm. Do not roll the sleeve since it can squeeze your hand and this can lead to false results.
- Use only clinically approved original cuff.
- If you want to follow the results of your blood pressure measurements, always perform measurements at the same time of day, since blood pressure changes during the course of the day.
- Measurements should be done after a 5 minute rest to ensure accuracy.

Fitting the Cuff

1. Plug the cuff connector to the cuff socket on the device left side.
2. Stretch your left (right) arm in front of you with your palm facing up. Slide and place the cuff onto your arm to let the air tube and artery mark region toward the lower arm. Wrap and tighten the cuff above your elbow. The artery mark on the edge of the cuff should be approx. 0.8" to 1.2" (2 cm to 3 cm) above your elbow. Align the tube over the main

- arteries on the inside.
3. Leave a little free space between the arm and the cuff: you should be able to fit two fingers between them. Remove all clothing covering or constricting the measurement arm.
 4. Press the hook material firmly against the pile material. The top and bottom edges of the cuff should be tightened evenly around your upper arm.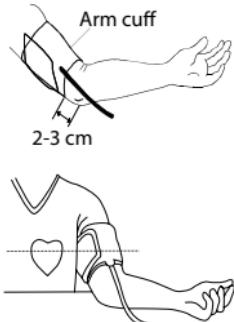


Measuring Procedure

Always apply the pressure cuff before turning on the monitor.


1. Press ON/OFF button. All icons appear two seconds on DISPLAY, then switch to measurement, and display «00».

2. Press the air valve to deflate until the «♥» disappears on the LCD screen.
3. Then press the inflatable ball repeatedly. After pressure value reaches 180 mmHg on the LCD display, or the «▼» appears on the LCD screen, stop pressing the inflatable ball. (The «▲» on the LCD screen indicates that it is important to press the inflatable ball until the «▼» appears on the LCD screen).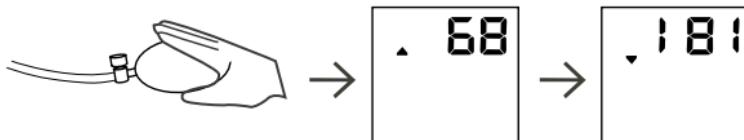

4. The pressure value decreases slowly on LCD. When the «♥» flashes on LCD, it means the device begins to measure blood pressure until the measure result is displayed on LCD. Then press the air valve to deflate until the «▼» disappears on the LCD screen.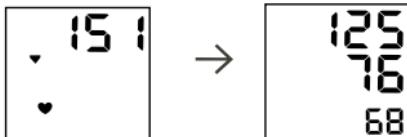



NOTE: The device stops measuring and the «» appears on LCD during measuring blood pressure. This situation indicates that it is important to press the ball to inflate until the «» disappears on LCD.

Reading Measurement Results

When the measurement has been concluded, the measured blood pressure and pulse rate and color bars classified by WHO (World Health Organization) are now displayed.

The appearance of this symbol  signifies that an irregular heartbeat was detected. This indicator is only a caution. It is important that you be relaxed, remain still and do not talk during measurements.


On the graphic indicator the element showing level of arterial pressure on a scale of World Health Organization is displayed.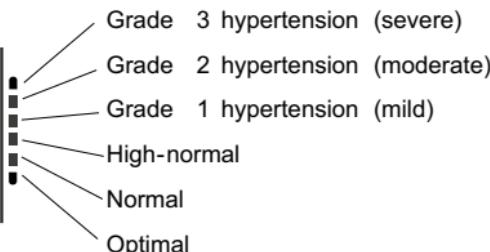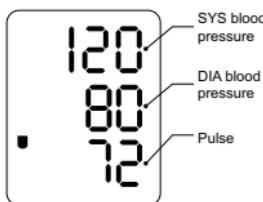

NOTE: To prolong the batteries life the device switches off automatically if no button is pressed for 1 min 30 sec. Otherwise you can switch it off by pressing the ON/OFF button.

Discontinuing a Measurement

If it is necessary to interrupt a blood pressure measurement for any reason (e.g. the patient feels unwell), the quick-release valve at the pumping bulb can be pressed at any time. The device then immediately lowers the cuff pressure automatically.

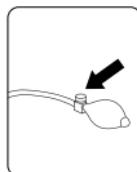MEMORY FUNCTION

Semi-automatic blood pressure monitor model M-150S automatically stores the result measured.

To see the result of the last measurement press and hold the ON/OFF button for at least 3 seconds.

If an irregular heartbeat interval was detected in the measurement stored in memory, the icon will be displayed.

ERRORMESSAGES/TROUBLESHOOTING

If you have trouble in using the unit please check the following points first.

ERROR DISPLAY	POSSIBLE CAUSE	HOW TO CORRECT
Nothing is displayed when you push the ON/OFF button or battery icon flashes	No battery installation	Insert batteries
	Battery worn out	Replace with new batteries
	The polarities of batteries placed wrongly	Insert battery in the correct polarities
E1: Can't normally increase pressure	Check your cuff for air leakage	Replace cuff with new one
E3: Inflate pressure too high		Remeasure or send back to dealer to recalibrate
E2 E4: Hand shaking while measurement	Hand or body shaking while measurement	Keep static position and measure again
Battery icon on	Battery low power	Replace battery and measure again
The systolic pressure value or diastolic pressure value too high	The cuff was held lower than your heart	
	The cuff was not attached properly	Keep correct position and measure again
	You moved or spoke during measurement	

The systolic pressure value or diastolic pressure value too low	The cuff was held higher than your heart You moved or spoke during measurement	Keep correct position and measure again
---	---	---

ATTENTION

Should any technical malfunctions arise in the blood pressure monitor, please contact the authorized service center of **Dr. Frei®**. **Never attempt to repair the instrument yourself!** Any unauthorized opening of the instrument invalidates all warranty claims.

NOTE: Blood pressure is subject to fluctuations even in healthy people. Please remember that comparable blood pressure measurements always require the same time and the same conditions! These are normally quiet conditions. If you follow the correct procedures described above and still get the fluctuations of blood pressure of more than 15 mmHg and/or you repeatedly hear irregular pulse tones please consult your doctor.

CARE AND MAINTENANCE

Do not expose the device to either extreme temperatures, humidity, dust or direct sunlight.	
Handle the cuff carefully and avoid all types of stress through twisting or buckling in order not to damage the sensitive air-tight bubble.	
Clean the device with a soft, dry cloth. Do not use gas, thinners or similar solvents. Spots on the cuff can be removed carefully with a damp cloth and soapsuds. Do not submerge the cuff in water!	

<p>Handle the tube carefully. Keep the hose away from sharp edges.</p>	
<p>Do not drop the monitor or treat it roughly in any way.</p>	
<p>Never open the monitor! Otherwise the manufacturer's calibration will be spoiled.</p>	

SYMBOL DESCRIPTION

SYMBOL	REFERENT
	Consult instructions for use
	Manufacturer
	Serial number
	Caution, consult accompanying documents
	Dispose of in accordance with the requirements of your country
	Type BF equipment
	Class II
CE 0413	CE mark

NOTE: According to international standards, your monitor should be checked for calibration every 2 years.

TECHNICAL SPECIFICATIONS

Measuring Method	Oscillometric
Indication	Digital LCD display
Measuring Range:	Pressure: (30~280) mmHg Pulse: (40~199) Beat/min
Accuracy:	Static Pressure: ± 3 mmHg. Pulse: $\pm 5\%$
Memory:	Last measurement memory
Power supply:	4x1.5 V batteries (LR6 or AA)
Operating condition:	+5°C~+40°C, 15% RH~93% RH Atmospheric pressure: 70 kPa~106 kPa
Storage condition:	-20°C~+55°C, 0% RH~93% RH Atmospheric pressure: 50 kPa~106 kPa
Dimensions:	Approx: 90(W) X 135(H) X 62(D) mm
Weight:	Approx: 340 g, excluding batteries
Classification	Type BF
Cuff size	22-32 cm

* The specifications and the outer design are subject to change for improvement reasons without previous notice.

WARRANTY

Your semi-automatic blood pressure monitor model M-150S is warranted for 5 years from date of purchase. Warranty for the cuff is 1 year from the date of purchase. The warranty does not apply to damage caused by improper handling, accidents, not following the operating instructions or self-maintained alterations made to the device.

The warranty is only valid upon presentation of the warranty card which was correctly filled in and sealed.

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For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

■ Shenzhen Pango Electronic Co., Ltd., No. 25 1st Industry Zone, Fenghuang Rd.,
Xikeng Village, Henggang Town, Longgang District, Shenzhen City, Guangdong Province,
China.  Lotus NL B.V., Koningin Julianaplein 10, 1e Verd, 2595AA, The Hague,
Netherlands.

global-swiss.ch
dr-frei.com

CE ISO
0413 13485