AUTOMATIC BLOOD PRESSURE MONITOR

M-PRO
USER MANUAL

dr-frei.com

	Contents of the transport package are fragile therefore it shall be handled with care.
	Indicates temperature limits within which the transport package shall be stored and handled.
	Set User, Voice, Unit
	Start measurement, Stop measurement, Turn off
	Memory icon, when the MEM icon flashing, operator could save or recall memories.
	IP21: Protected against solid foreign objects of 12.5 mm ø and greater; Protection against vertically falling water drops.
	Non-ionizing electromagnetic radiation
	Production lot number
	Production date
	Manufacturer
	The device should not be used after the end of the shown or the day
	CE 6123: CE mark: indicates that the device complies with the EU 2017/745.
	EEC REP: Authorized Representative in the European Community
	MD: Medical Device

UDI template:
(01) (Device Identification)
(17) Expiration Date
(11) Production Date
(10) Lot Number
(21) Serial Number

7

ADAPTER USE (OPTIONAL)

1. The optional AC adapter should comply with the requirements of IEC 60601-1, and all configurations should meet the requirements of the medical electrical system. Please note that local laws take precedence over the above requirements. In case of any doubt, please consult your local representative or technical support. 2. When using AC power, select the power cord of the TYPE-C jack, and plug it into the power jack. 3. To remove the AC adapter, first unplug the adapter from the outlet and then disconnect the power cord from the device outlet.

Adapter parameter requirements:

Input voltage: DC 5V, 1A

Maximum output current: at least 1A

TYPE-C socket

Note: do not insert the adapter when using the product.

Note
1.Charging may take 3-4 hours.
2.Every 2 months (or when battery life is significantly shorter), fully charge the monitor, then allow the battery to drain until the monitor shuts off. This will optimize battery performance.
3.Battery life depends on frequency and time of use. If battery life is unusually short, please contact your customer support team.
4.Only use a Type-C USB cable to recharge the monitor.
5.In extreme conditions, the battery may leak corrosive juice. If this comes into contact with eyes or skin, rinse immediately with water and seek medical attention. Do not replace the lithium battery by yourself, inadequately trained personnel could result in a hazard.

SAFETY INFORMATION

1. General usage
Do not adjust measurement based on measurement values from that blood pressure monitor, please refer to your physician. Only a physician is qualified to diagnose and treat High Blood Pressure.

• The monitor is not intended to be a diagnostic device.

2. General precautions
• Do not forcibly crease the arm cuff or the air tube excessively.

• Do not press the air tube or the air tube excessively.

• Do not inflate the arm cuff when it is wrapped around your arm.

• Do not use the device outside the specified environment. It may cause an inaccurate reading.

3. Cleaning the monitor and cuff
• Please do not clean the monitor or the cuff with strong detergents or solvents.

• Please do not clean the monitor or the cuff with strong detergents or solvents.

4. Disposal
• Do not dispose of electrical appliances as unsorted municipal waste, use separate collection facilities. Contact your local government for information regarding the collection systems available. If electrical appliances are disposed of in landfills or dumps, hazardous substances can leak into the groundwater and get into the food chain, damaging your health and well-being.

5. Manufacturer information:
Manufacturer: Allcon Medical Shenzhen, Inc
Address: Room 4C, Building 1, Zhongcheng Biomedical Industrial Park, No. 21 Linhu Road, Jinsha Community, Kengzi Street, Pingshan District, 518118 Shenzhen, Guangdong, PEOPLE'S REPUBLIC OF CHINA
Tel: 0086-755-26501548 Fax: 0086-755-26504849
Email: Info@allcon-med.com Website: http://www.allcon-med.com

6. Periodic safety checks
If you use the device with power adapter, preventive inspection and maintenance to be performed at least once every 3 months.

• Every time before use, please check the adapter, once damaged, never to use.

• Please clean the plug of adapter plug at least once a year. Too much dust on plug may cause the fire.

The manufacturer reserves the right to make technical changes without notice in the interest of progress.

Prior notices will not be given in case of any amendments within this manual. The mentioned trademarks and names are owned by the corresponding companies.

13

14

Test Frequency (MHz)	Band (MHz)	Service	Modulation	Modulation (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)
385	380 - 390	TETRA 400	Pulse modulation	1.8	0.3	27
450	380 - 390	GMRS 460, FRS 460	FM ±5kHz	2	0.3	28
710	704 - 787	LTE Band 13, 17	Pulse modulation	0.2	0.3	9
745			217 Hz			
780						
810	800 - 960	GSM 800/900, TETRA 800, CDMA 850, LTE Band 5	Pulse modulation	2	0.3	28
870			18 Hz			
930						
1720	1700 - 1990	GSM 1900, CDMA 1900, GSM 2100, DECT, IEEE 802.11, 25. UMTS	Pulse modulation	2	0.3	28
1845			217 Hz			
1970						
2450	2400 - 2570	Bluetooth, WLAN, 802.11, 2450, LTE Band 7	Pulse modulation	2	0.3	28
5240	5100 - 5800	WLAN 802.11 a/n	Pulse modulation	0.2	0.3	9
5785			217 Hz			

PERIODIC SAFETY CHECKS

If you use the device with power adapter, preventive inspection and maintenance to be performed at least once every 3 months.

• Every time before use, please check the adapter, once damaged, never to use.

• Please clean the plug of adapter plug at least once a year. Too much dust on plug may cause the fire.

The manufacturer reserves the right to make technical changes without notice in the interest of progress.

Prior notices will not be given in case of any amendments within this manual. The mentioned trademarks and names are owned by the corresponding companies.

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DEAR DR. FREI AUTOMATIC BLOOD PRESSURE MONITOR

Thank you for choosing Automatic Blood Pressure Monitor Dr. Frei™ M-PRO. We are sure that having appointed worthily the high quality and reliability of this device you will become a regular user of the products of Trademark Dr. Frei™.

Before starting to use this device please study the user manual carefully. The user manual offers all information you need to correctly measure your blood pressure and pulse. For all questions concerning the device please contact your local distributor or Dr. Frei™ service center in your country.

ATTENTION

This blood pressure monitor is designed to carry out self-control over blood pressure but NOT to make self-diagnosis of hypertension/hypotension. Please DO NOT diagnose by yourselves basing on the measurement results obtained with the blood pressure monitor. Please DO NOT execute self-treatment of high/low blood pressure and DO NOT change the methods prescribed without consulting your doctor.

Type BF applied part.

Read the instructions carefully before using this device.

2

TABLE OF CONTENTS

What Is Blood Pressure?	4
What Values Are Normal?	5
Intended Use / Indications for Use	6
Package contents	6
Explanation of Marks or Symbols	6
Specification	8
Kids mode	9
Tie the cuff	10
Start the Measurement	11
Recall records	11
Delete record	12
Battery replacement, battery charging	12
Adapter use (optional)	13
Safety Information	13
Tips for operation	15
Common Q&A on blood pressure	15
Care and maintenance	15
Trouble Shooting	16
Electromagnetic Compatibility	16
Periodic safety checks	16
Warranty	20

IMPORTANT INFORMATION ON BLOOD PRESSURE AND ITS MEASUREMENT

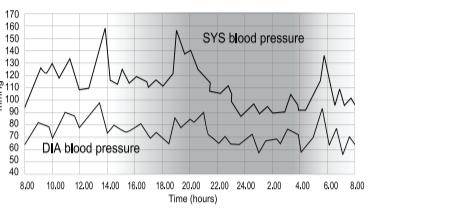
What Is Blood Pressure?

Blood pressure is the pressure that your blood extends to the vascular walls. Blood pressure is necessary for the normal functioning of the heart and body. Thanks to it the cells get oxygen that provides them with energy. The heart performs the function of a "pump", sending blood to the blood vessels. The heart beat creates a certain level of the blood pressure.

There are 2 kinds of blood pressure: systolic and diastolic.

Diastolic pressure is subject to fluctuations during the day even in healthy people. The fluctuations are influenced by a number of factors - time of day, person's condition, physical or mental activity, environment, etc.

Day-Night Fluctuations of the Blood Pressure
An increase of blood pressure increases the burden onto the heart, affects blood vessels making their walls thicker and less elastic. One of the features of the hypertension is its ability to remain unnoticed for a long time.

The patient at his early stages, that's why the self-control of the blood pressure is so important. With the illness progressing, headaches and regular dizziness appear, the sight declines, the functioning of vessels (encephalon, heart, kidneys, blood vessels) breaks down. Without special treatment the complications of hypertension might be kidney damages, breast-pang, paralytic stroke, aphasia, dementia, heart attack and stroke.

4

Which Values are Normal?

The world standard as for the norms of the blood pressure is the Classification* of the World Health Organization (WHO):


Table 1. Definitions and classification of blood pressure levels (mmHg)

Category	Systolic	Diastolic
Optimal	< 120	< 80
Normal	< 130	< 85
High-normal	130 - 139	85 - 89
Grade 1 hypertension (mild)	140 - 159	90 - 99
Subgroup: borderline	140 - 149	90 - 94
Grade 2 hypertension (moderate)	160 - 179	100 - 109
Grade 3 hypertension (severe)	≥ 180	≥ 110
Isolated systolic hypertension	≥ 140	< 90
Subgroup: borderline	140 - 149	< 90

When a patient's systolic and diastolic blood pressures fall into different categories, the higher category should apply.

* printed with curtailments

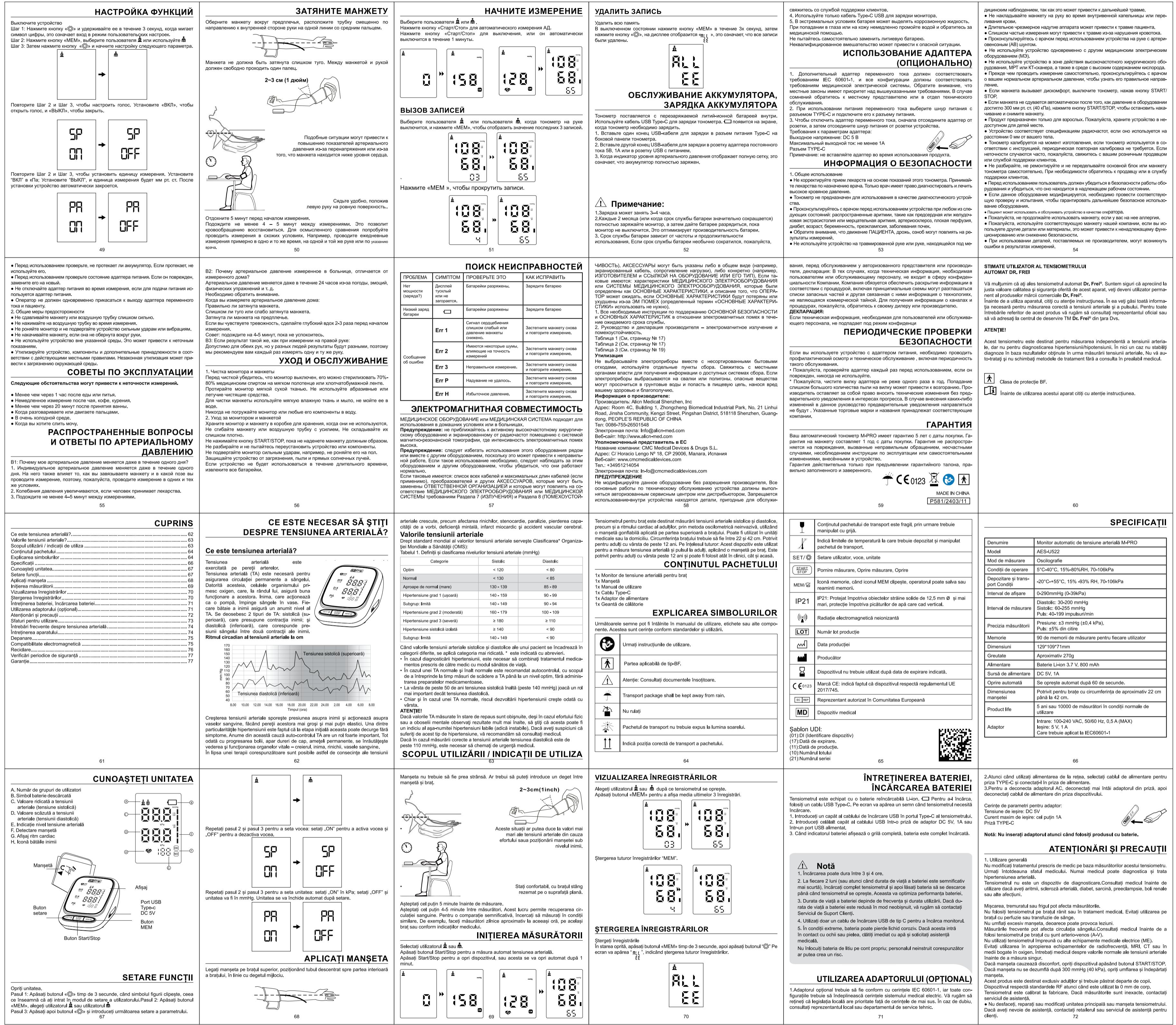The diagnosis of hypertension requires

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