**COMPRESSOR
NEBULIZER**
TURBO CARE
USER'S MANUAL

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EN

DEAR DR. FREI COMPRESSOR NEBULIZER OWNER

Thank you very much for purchasing Nebulizer. Be sure to read this Instruction Manual before using the unit for you to use it safely and correctly.

COMPRESSOR NEBULIZER

BEFORE USING THE UNIT	
Notes on Safety	3
Product Introduction	5
Product Contents.....	6
Name of the Parts.....	7
HOW TO ASSEMBLE	
How to Prepare the Power Source.....	8
How to Fill the Medication	8
Assemble the Medicine Cup.....	9
Assemble the Air Tube.....	10
HOW TO USE THE UNIT	
How to Use the Unit	11
HOW TO CLEAN, DISINFECT AND MAINTAIN THE DEVICE	
How to Clean the Unit after Inhalation.....	13
How to Disinfect	14
Maintenance of the Device	15
TROUBLESHOOTING	
Troubleshooting problems	16
Specification	17

- This unit is a medical instrument. Be sure to follow the instructions of a doctor and use the unit correctly. For the type, dose, and regimen of medication, be sure to follow the instructions of a doctor.
- The nebulization characteristics of this unit differ by the properties of the medication.
- Especially with the use of a medication having a high surface activity or viscosity such as a medication solubilizing agent or expectorant, the nebulization rate may be reduced. The nebulization rate may also be reduced when the temperature of the medication is low.

NOTES ON SAFETY

Be sure to read the notes on safety before using the unit.

The warning signs and the sample icons shown here are listed for you to use this product safely and correctly as well as to prevent the risk and the damage to you and others from happening. The icons and meanings are as follows.

Warning

For the type, dose, and regimen of the medication, be sure to follow the instructions of a doctor.

When you use the unit for the first time after purchasing it or after not using it for a long period of time, be sure to clean and disinfect the Medicine Cup, Inhalation Mask Mouthpiece.

Clean and disinfect Medicine Cup, Inhalation Mask, and Mouthpiece after each use.

Be sure that the patient is an intended operator.

Don't maintain the device while it is in use with the patient.

Be sure to dry the cleaned and disinfected parts promptly, and store them in a clean place.

Do not place or attempt to dry the device, components, or any of the nebulizer parts in a microwave oven.

Keep the device out of the reach of unsupervised infants and children. The device may contain small parts that can be swallowed, moreover strangulation due to cables and hoses may occur.

Don't use accessories such as medicine cup, mask, nosepiece, mouthpiece that is not provided by the supplier described in the brochure.

Do not wash the Main Unit and Power Plug with water or hot water.

Do not immerse the Main Unit in water or other liquid.

Do not cover the compressor with a blanket, towel, or any other type of cover during use. This could result in the compressor overheating or malfunctioning.

Do not use the device where the device may be exposed to flammable gas or vapors.

Pentamidine is not an approved medication for use with this device.

Always dispose of any remaining medication in the medicine cup after each use.

Do not leave the device or its parts where it will be exposed to extreme temperatures or changes in humidity, such as leaving the device in a vehicle during warm or hot months or where it will be exposed to direct sunlight.

Do not use or store the device where it may be exposed to noxious fumes or volatile substances.

Make sure that the Medicine Cup is clean before use.

Do not use in an anesthetic or ventilator breathing circuits.

Not suitable for use in the presence of a flammable anesthetic mixture with air, oxygen, or nitrous oxide.

Risk of Electrical Shock: Do not use the compressor (main unit) or Power Plug if they are wet.

Do not plug or unplug the Power Plug into the electrical outlet with wet hands.

Do not use or store the device in humid locations, such as a bathroom. Use the device within the operating temperature and humidity.

Do not operate the device with a damaged power cord or plug.

Maintenance and Storage Do not leave the cleaning solution in the nebulizer parts. Rinse the nebulizer parts with clean hot tap water after disinfecting.

Store the device and the components in a clean, safe location.

If you have any problems with this device, such as setting up, maintaining or using, please contact with service local service Don't open or repair the device by yourself.

NOTES ON SAFETY

Safety Be sure to read the notes on safety before using the unit.

The warning signs and the sample icons shown here are listed for you to use this product safely and correctly as well as to prevent the risk and the damage to you and others from happening. The icons and meanings are as follows.

Caution

Do not inhale water in the Medicine Cup.

Do not spill water or other liquids on the compressor and Power Plug. These parts are not waterproof. If liquid spills on these parts, immediately unplug the Power Plug and wipe off the liquid with gauze or other soft absorbent material.

Do not drop or apply strong shock to the Main Unit or Medicine Cup.

Do not disassemble, repair, or modify the unit.

Provide close supervision when this device is used by, on, or near infants, children, or compromised individuals.

If the device is used continuously, the service life of the device may be shortened.

Limit use to 30 minutes at a time, and allow a 20-minute interval before using the device again.

Do not insert any object into the compressor.

Make sure the Air Filter is clean. If the Air Filter has changed color or has been used for more than 60 days, replace it with a new one.

Make sure the Medicine Cup is correctly assembled, the Air Filter is properly installed, and the Air Tube is correctly connected to the compressor and the Medicine Cup. Air may leak from the Air Tube during use if not securely connected.

Do not use the device if the Air Tube is bent.

Do not add more than 8mL of medication to the Medicine Cup.

Do not operate the device at temperatures higher than +40°C (+104°F).

Do not tilt the Medicine Cup so the angle of the kit is bigger than 45°. Medication may flow into the mouth.

Do not shake the Medicine Cup while using the device.

Use only authorized parts and accessories. Parts and accessories not approved for use with the device do not perform the expected specification or it may damage the unit. To avoid injury to the nasal mucosa, do not squeeze the optional Nose piece into the back of the nose.

When using this device, there will be some noise and vibration caused by the pump in the compressor.

There will also be some noise caused by the emission of compressed air from the Medicine Cup. This is normal and it does not indicate a malfunction.

When sterilizing parts by boiling, make sure that the container does not boil dry.

Do not use the device while sleeping.

Remove the Power Plug from the device after use.

Unplug the Power Plug from the electrical outlet before cleaning the device.

Do not leave the Medicine Cup with medication after using it.

Do not store the Air Tube with moisture or medication remaining in the air tube. This could result in infection as a result of bacteria.

Do not wrap the Power Plug around the compressor (main unit).

People who are not trained with specified skills are not permitted to maintain the device.

PRODUCT INTRODUCTION

This product is intended for the treatment of respiratory diseases such as asthma, allergies, etc. Working principle: The compressor forces air to the nebulizer. When the air enters the nebulizer, it converts the prescribed medication into an aerosol of microscopic droplets that can easily be inhaled.


The meaning of the symbols used in this instruction brochure.

SYMBOLS

SYMBOLS	MEANING
	The symbol for caution. This is a safety symbol that is used to highlight that there are specific warnings or precautions associated with the devices that are not otherwise found on the label. The symbol can also be used to mean» attention, see Instructions for Use.”
	Manufacturer
	The symbol indicates that the device should be sent to the special agencies according to local regulations for separating collection after its useful life.
	The symbol for “THE OPERATION GUIDE MUST BE READ”; Indicates the need for the user to consult the instructions for use.
	This symbol indicates that the device includes EC 60601-1 Type BF Applied Part.
IP21	Protection against ingress of water, it means the device could protect against vertically falling water drop 15°.
	Indicates the date when the medical device was manufactured.
	This symbol shall be accompanied by the manufacturer’s serial number.
	The symbol for lot number; Indicates the manufacturer’s batch code so that the batch or lot can be identified.
	Indicates the Authorized representative in the European Community.
	The symbol for CE Mark. This symbol certifies that a product has met European Union consumer safety, health, or environmental requirements.

PRODUCT CONTENTS


The following items are contained in the box. If any item is missing, please contact the store where you purchased the product or the nearest dealer.

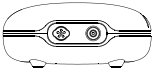Compressor	Medicine Cup	Inhalation Mask (Adult)
		
Inhalation Mask (Child)	Mouthpiece	Air Tube
		
Air Filters	Type-C Cable	
 X 5		

(Refer to Pages 7, 8, 9.)

To start inhalation, you must assemble the unit.
Please read pages 7, 8, and 9 and understand the points for assembling before starting to use the unit.

NAME OF THE PARTS

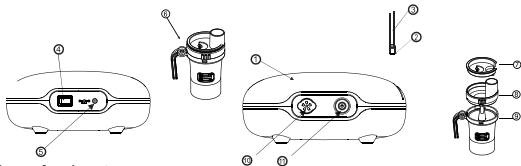Name of each part

1. Compressor (Main Unit)	7. Medicine cup cover
2. Air outlet	8. Medicine cup upper shell
3. Air Tube	9. Medicine cup lower shell
4. Power Switch	10. Air filter and cover
5. Type-C Cable	11. Tube connector
6. Medicine cup	

HOW TO PREPARE THE POWER SOURCE

Make sure of that the mouthpiece, nosepiece, mask and air filter be clean before use

1. Prepare the Power Source

Connect the DC adapter to the DC Jack of the Main Unit, and then to the power

2. Fill in the Medication

Lift the Mouthpiece, or Nosepiece or Mask from the Medicine Cup, and unscrew the Nebulizer Upper Shell.


Fill in the correct amount of prescribed medication in the Nebulizer Bottom Shell.


ASSEMBLE THE MEDICINE CUP

3. Assemble the Medicine Cup


A. Screw the Nebulizer Upper Shell to the Nebulizer Bottom Shell clockwise until it is securely closed.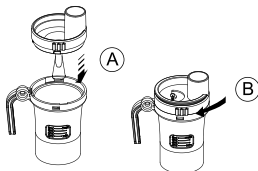

B. Assemble the desired inhalation accessories.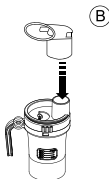

Assemble the Mouthpiece

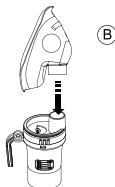
Assemble the Mask

ASSEMBLE THE AIR TUBE

4. Assemble the Air Tube

Step 1: Twist the Air Tube plug and push it firmly into the Air Tube Connector of the compressor.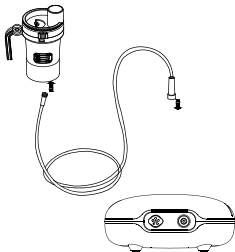

Caution: Keep the Medicine Cup upright when assembling the Air Tube. Otherwise, medication may spill out of the Medicine Cup.



Caution: Don't use the Unit if the Air Tube is bent.

Step 2: Twist the Air Tube plug slightly and push it firmly into the bottom of the Medicine Cup. Use the Medicine Cup Holder as a temporary holder for the Medicine Cup.

HOW TO USE THE UNIT

1. Hold the Medicine Cup as shown in the Figure

A. Screw the Nebulizer Upper Shell to the Nebulizer Bottom Shell clockwise until it is securely closed.

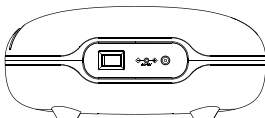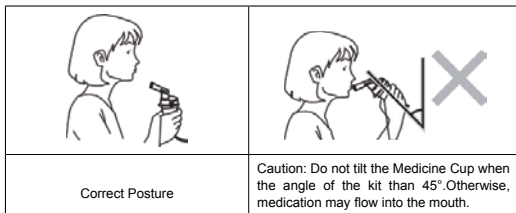2. Turn on the Unit

Press the power button for 0.5 seconds, when the compressor starts, nebulization begins.

HOW TO USE THE UNIT

3. Inhale medication as instructed by the Physician.

The difference among the inhalation accessories as following.

Mouthpiece - Insert the Mouthpiece into your mouth.

Inhale medication normally. Exhale normally

through the Mouthpiece.

3. Mask

- Place the Mask over the nose and mouth. Pull the elastic

strap over the head. Gently pull on the strap to secure the

mask over the nose and mouth. Inhale the medication.

Exhale normally through the mask.

Nosepiece (Optional)

- Inhale the medication through the nosepiece as illustrated.

Exhale through the mouth.

4. Turn off the Unit

Press power button for 0.5 seconds when the compressor stops, nebulization stops.

HOW TO USE THE UNIT

5. Disconnect the Air Tube from the Medicine Cup

- Hold the Air Tube Plug and gently pull it down - Check the Air tube. Make sure that No condensation or moisture remain in the Air Tube.

If any condensation or moisture remains in the Air tube, remove the moisture from the air tube. Follow the directions below: 1. Make sure the Air Tube is still connected to the air tube connector of the compressor. 2. Turn on the Unit, and the compressor will start and pump air through the air tube to expel the moisture. 3. Turn off the Unit.

6. Disconnect the air tube from the compressor

- Hold the Air Tube Plug and gently pull the Air Tube Plug off the Air Tube Connector on the compressor.

7. Unplug the Type-C cable from the power outlet

- Unplug the DC Adapter from the power outlet and Type-C of the main unit.

HOW TO CLEAN THE UNIT AFTER INHALATION

Be sure to clean and store the unit after inhalation.

1. Remove the inhalation accessory from the Medicine Cup.
2. Disconnect the Air Tube from the Medicine Cup.
3. Gently screw off the Medicine Cup Upper Shell.
4. Discard the remaining medication.

5. Wash the parts sufficiently in water.
6. Hand dry or air dry in a clean environment using a soft, clean, lint-free cloth.

7. Assemble the Medicine Cup and store it in a dry bag.


Caution: Clean the casing of the main unit by using a soft cloth moistened with water or mild detergent.

Do not use abrasive cleaners.

HOW TO DISINFECT

Disinfect the Medicine Cup and Mouthpiece or nosepiece after the last treatment of the day.

- The method as described below:

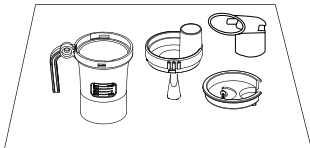Use a commercially available disinfectant with the instructions provided by the disinfectant manufacturer, it applies to the masks' disinfection.

- Submerge the parts in the disinfectant solution for the specified period. - Remove the parts and discard the solution.

- Rinse the parts with clean hot tap water, shake off excess water, and dry when in a clean environment.

MAINTENANCE OF THE DEVICE

USE

This unit is a medical instrument. Be sure to follow the instructions of a doctor and use the unit correctly.

The nebulization characteristics of this unit differ by the properties of the medication.

Especially with the use of a medication having a high surface activity or viscosity such as a medication solubilizing agent or expectorant, the nebulization rate may be reduced. The nebulization rate may also be reduced when the temperature of the medication is low.

Do not use the device if it is damaged in any way. The continuous use of a damaged unit may cause injury, improper results, or serious risk.

When not in use, store the device in a dry room and protect it against extreme moisture, heat, lint, dust, and direct sunlight. Never place any heavy objects on the storage case.

Provide close supervision when the device is used by, on, or near infants, children, or compromised individuals.

No modification of this equipment is allowed.

Inhalation Mask is recommended when the main unit is used by children under 5.

Keep the device out of the reach of unsupervised infants and children. The mouthpiece or other small parts may be swallowed.

A metallic sound may be generated, but it is not a trouble.

The device shall be disconnected from the power source after use.

The mouthpiece and inhalation mask contact the user directly, please keep it clean.

STORAGE

DO NOT store the device in locations exposed to high temperatures (over 60°C), low temperatures (below -20°C), high relative humidity (over 90%), or excessive amounts of dust.

Never place any heavy objects on the storage case.

Store the device and component in a clean, safe location. Keep the unit out of reach of small children.

MAINTENANCE

DO NOT drop the device or subject it to other shocks or vibrations.

Never dry the parts in the microwave oven.

Be noted that parts other than the mouthpiece cannot be boiled!

Do not carry or leave the nebulizer with medication in the medicine cup.

- Never dry the parts in the microwave oven.
- Never use an autoclave, EOG gas disinfection, or low-temperature plasma sterilizer to disinfect the device.

TROUBLESHOOTING PROBLEMS

Trouble	Possible Cause	How to Correct
No power to the device when pressing the switch button	The DC adapter is not plugged into the main unit Or the battery is powered off	Plug the DC adapter into the DC jack, and press the switch button.
No nebulization or low nebulization rate when the power is on.	No medication in the Medicine Cup. Too much or too little medication in the Medicine Cup.	Add the correct amount of prescribed medication to the Medicine Cup.
	The Medicine Cup is not correctly assembled.	Make sure the Medicine Cup is correctly assembled and the inhalation accessory is correctly attached.
	The nozzle is blocked.	Clean and disinfect the Medicine cup to remove the blockage.
	The Medicine Cup is tilted at an incorrect angle	Hold the Medicine Cup correctly. Do not tilt the Medicine Cup so the angle of the kit is bigger than 45 degrees.
	The Air Tube is incorrectly attached.	Make sure the Air Tube is correctly attached to the compressor and the medicine Cup.
	The Air Tube is folded or damaged. The Air Tube is blocked	Make sure the Air Tube is not folded, kinked, or bent Inspect the Air Tube for any damage. Replace the Air tube if damaged
	The Air Filter is dirty.	Replace the Air Filter with a new clean Air Filter.
The device is very hot.	The compressor is covered.	Do not cover the compressor with any type of cover during use.
	Operating continuously over 30 minutes.	Limit use to 30 minutes at a time and allow a 20 minutes interval before using the device again.
The device is abnormally loud.	The Air Tube is incorrectly attached.	Attach the Air Filter cover correctly. Make sure the Air Filter cover is not blocked.

Note: If the unit does not nebulize normally after taking the procedure mentioned above, contact the store where you purchased the unit or the nearest dealer.

SPECIFICATION

Product Name	DC Compressor Nebulizer
Model	Turbo Care (CDC200)
Standard accessories	Medicine cup, Air Tube, Adult Mask, Child Mask, Mouthpiece, Filters(5)
Power source	5V / 2A
Nebulization Pressure	40KPa-80KPa
Working Environment	Temperature: 5 to 40°C Humidity: 15 to 80%RH Atmospheric Pressure: 70.0KPa-106.0KPa
Storage Environment	Temperature: -20 to 60°C Humidity: 10to5%RH Atmospheric Pressure: 70.0KPa-106.0KPa
Device type (component/sub-assembly/ equipment /system)	Equipment
Sound Level	≤55dB(A)
Capacity of Medicine Cup	8mL
Nebulization Rate	≥0.2mL/ min
Particle Size MMAD	≤3.8 μ m±25%
Dimensions	157x98x69mm
Weight (not including battery)	Approximately 250g (not including Medicine Cup and Air Tube)
Protection against harmful ingress of water or particulate matter	IP21

- The nebulization rate is lowered at low temperatures.
 - Specifications and appearance may be changed without prior notice for improvement.
- limit use to 30 minutes at a time and allow a 20 minute interval before using the device again

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